

HEMOSTAB-FE
Liquid for capillary hemostasis

Indications:

- arresting capillary bleeding;
- capillary bleeding of the gums in the cervical area of the tooth;
- apical bleeding in the root canal.

Composition

Ferrous sulphate
Hydroxyquinoline sulfate
Distilled water

Properties

“Hemostab” is an aqueous solution of ferrous sulphate. Ferrous sulphate stops capillary bleeding. In the case when it is necessary to arrest bleeding from the gingiva, it is necessary to isolate treated area and apply hemostatic solution to the gingival surface area of bleeding.

Pulp extirpation may cause bleeding in the root canal, hematoma formation at the apex of the tooth root following the inflammatory process with subsequent development of granuloma. In order to alleviate such effects a root canal should be treated with turundas moistened with a liquid hemostatic. In a short time the liquid stops and prevents recurrent bleeding, which gives the opportunity to continue working in the root canal.

Package contents and storage

Liquid is available in bottles of 13 ml.
Keep in cool dry place.
Shelf life is 3 years.