

**ALVOSTASE (sponge №1)**  
**Hemostatic and antiseptic alveolar dressing**

**Indications**

For the prevention and treatment of alveolitis. It is introduced into extraction site.

**Composition and properties**

Tricalcium phosphate  
Eugenol  
Olive oil  
Hemostatic sponge  
Iodoform

"Alvostase" (sponge №1) is a special agent for the prevention and treatment of alveolitis. Placed in the alveolus "Alvostase" (sponge №1) quickly reduces inflammation and pain. Material exerts its effect in a few hours, and then gradually resorbed in the extraction site.

Preparation does not cause inflammation of the oral mucosa. When using the material as a prophylactic remedy preparations in a very short time contributes painless healing of the extraction site.

**Contraindications.**

"Alvostase" (sponge №1) is contraindicated in patients with hypersensitivity to iodine.

**Recommended use**

After extraction of a tooth it is necessary to wash and clean the extraction site and to achieve the formation of a blood clot. If necessary, in case of the possible inflammation, it is recommended to insert "Alvostase-sponge" to the extraction site. Then cover it by a dry cotton tampon. In case of traumatic extraction the site can be sutured over the sponge. The sponge is resorbed completely.

Only for topical application.

**Package contents and storage**

Material is packed in jars with 30 sponges 1x1 cm, impregnated by curing paste.  
Keep in cool dry place.  
Shelf life is 3 years.