

POLISHPASTE-Z + F

Paste for removal of soft dental plaque and fine polishing of dental enamel

Indications:

- removal of soft plaque with instruments;
- final polishing of tooth structures after removal of dental calculus;
- fluoridation of tooth enamel;
- antiseptic treatment of the enamel.

Composition

Fine abrasive particles
Binding agents
Fluoride-containing components
Filler
Silicone additives
Flavoring agent

Properties

The action of "Polishpaste-Z +F" is based on the specific properties of the fine abrasive particles with the addition of fluoride-containing components. Because of these properties "Polishpaste-Z +F" removes soft dental plaque without damage of the enamel.

Recommended use

Apply to the polishing cup or brush the required amount of "Polishpaste-Z +F". The procedure should be at medium speed without excessive pressure on the surface. Polish until the surface has the required shine. After the end of the polishing, wash out residual paste with water.

After plaque removal it is necessary to eliminate all kinds of irregularities that prevent the formation of new dental deposits. Final polishing is to bring to the surface perfectly smooth and it is advisable to cover the surface with fluoride varnish ("Fluorvarnish").

Package contents and storage

"Polishpaste-Z +F " is available in plastic jar of 40 g.
Recommended temperature of storage is +4°C to +24°C.
Shelf life is 3 years.