

DEHYDROL

Liquid for dehydrating and degreasing of root canals

Composition

Ethyl acetate
Acetone
Medical antiseptic solution 95%

The proposed material is intended for quick drying and degreasing of root canals and cavities before filling, as well as for the treatment of the teeth prepared for the placement of the artificial crown before fixation of prostheses.

Fluid does not prevent the polymerization of composite restorative materials.

Liquid is economic in use and helps in solving many problems. The preparation is not intended to remove fat from the gums.

Recommended use

Treat the cavity before filling the tooth with a cotton tampon dipped in the liquid. Wait a few seconds to complete evaporation of the liquid.

Package contents and storage

Packed in bottles of 13 ml or 25 ml of liquid.
Close the bottle after each use.
Keep the material in cool dry place.
Shelf life is 3 years.