

ALGISTAB
Hemostatic and antiseptic dental material

Indications

"Algistab" is used as a hemostatic agent after teeth extractions, removal of dental plaque, curettage of periodontal pockets, removal of exostosis of the alveolar processes, gingivectomy, microtraumas of gums after making dental impressions. Patients with hemophilia should receive specific treatment.

Composition

Alginat sodium
Potassium sorbate
Sodium benzoate
Iodoform
Tricalcium phosphate

Recommended use

Shake before use.

In all cases, except teeth extraction, remove the excess of blood with hydrogen peroxide or distilled water. Treat a bleeding wound with hygroscopic tampon, removing the excess of blood. Apply powder "Algistab" to the treated surface of the wound by spraying. Make sure that the powder covers the edges of the wound preventing the leakage of blood around the edges. In severe cases, apply the powder (by spraying) on the surface of a gauze tampon and apply to the surface of a bleeding wound.

Contraindication

The drug is contraindicated in patients with hypersensitivity to iodine.

Package contents and storage

Liquid is packed in plastic bottle with a tip for spraying. It contains 10 g of powder.
Keep in cool place protected from moisture.
Shelf life is 3 years.