

## **ALVOSTASE (filaments)**

### **Hemostatic and antiseptic alveolar dressing**

#### **Indications**

For the prevention and treatment of alveolitis.

#### **Composition and properties**

Tricalcium phosphate  
Eugenol  
Olive oil  
Viscose filament  
Iodoform

"Alvostase" (filaments) is a special agent for the prevention and treatment of alveolitis. Placed in the alveolus "Alvostase" (filaments) quickly reduces inflammation and pain. Material exerts its effect in a few hours, and then gradually discharged to the outside. With weak local toxicity, it does not cause inflammation of the oral mucosa. When using the material as a prophylactic remedy impregnation preparations in a very short time contributes painless healing of the extraction site.

#### **Contraindications.**

"Alvostase" (filaments) is contraindicated in patients with hypersensitivity to iodine.

#### **Recommended use**

After extraction of a tooth it is necessary to wash and clean the extraction site. A small amount of filaments carefully placed in the hole. "Alvostase" can be covered by a dry cotton tampon. Patients should be warned that over time a tampon will be gradually torn away.

Only for topical application.

#### **Package contents and storage**

The jar contains 10 g of viscose filament 1 cm of width and 1 m of length, impregnated by curing paste.  
Keep the material in cool dry place.  
Shelf life is 3 years.