

POLISHPASTE-D-FINISH
Paste with diamond filler for final polishing of composites and ceramics

Indications

Polishing paste for final polishing (last stage) of fillings made of composite materials light- and self-cured and ceramics.

Composition

Diamond abrasive
Water soluble polymer base
Silicone additives
Stabilizer
Flavoring agent

Properties

The action of "Polishpaste-D-Finish" based on the specific properties of the fine abrasive diamond powder.

Because of these properties diamond abrasive produces final polishing of fillings made of composite materials.

Recommended use

Before starting the procedures the surface should be slightly dried. Apply to the polishing cup or brush the required amount of "Polishpaste-D-Finish". The procedure should last 1-3 min at medium speed, not exceeding 8-10 thousands rpm and without excessive pressure on the surface. Polish until the filling has the required gloss. After the end of the polishing wash out residual paste with water.

Package contents and storage

"Polishpaste-D-Finish" is available in 2 syringes by 3 ml.
Recommended temperature of storage is +4°C to +24°C.
Shelf life is 3 years.