

POLISHPASTE-Z + W

Paste for whitening and fine polishing of dental enamel

Indications:

- final polishing of tooth enamel after removal of dental calculus;
- removal of soft plaque with instruments;
- whitening the tooth enamel.

Composition

Fine abrasive particles
Carbamide peroxide
Kaolin
Zinc oxide
Titanium dioxide
Silicone additives
Filler
Flavoring agent

Properties

The action of "Polishpaste-Z +W" is based on the specific properties of the fine abrasive particles and whitening abilities of carbamide peroxide. Because of these properties "Polishpaste-Z" removes soft dental plaque without damage of the enamel. Combination of aminofluoride with potassium nitrate contributes to the saturation of enamel by fluoride ions and elimination of enamel sensitivity.

"Polishpaste-Z +W" removes soft plaque without damaging the enamel, makes teeth whitening, allows to achieve maximum aesthetic effect.

Recommended use

Apply to the polishing cup or brush the required amount of "Polishpaste-Z +W". The procedure should be at medium speed without excessive pressure on the surface. Polish until a desired result. After the end of the polishing wash out residual paste with water.

Package contents and storage

"Polishpaste-Z +W" is available in plastic jar of 40 g.
Recommended temperature of storage is +4°C to +24°C.
Shelf life is 3 years.