

HEMOSTAB-AL
Liquid for capillary hemostasis

Indications:

- capillary bleeding of the gums in the cervical area of the tooth;
- intracanal bleeding.

Composition

Aluminum chloride
Hydroxyquinoline sulfate
Stabilizer

Properties

The liquid is a solution of aluminum chloride. Aluminum chloride stops capillary bleeding.

In the case when it is necessary to eliminate bleeding from the gingiva, it is necessary to isolate treated area and apply hemostatic solution to the gingival surface area of bleeding and around to avoid blood leakage.

In order to eliminate intracanal bleeding a root canal should be treated with tampons soaked in a solution of liquid hemostatic. In a short time the liquid stops and prevents recurrent bleeding, which gives the opportunity to continue working in the root canal.

Package contents and storage

Liquid is available in bottles of 13 ml and 25 ml.
Keep in cool dry place.
Shelf life is 3 years.